

switch sticks.®

www.switchsticks.com

Seat Stick Instructions

Weight Capacity: 220 lbs.

Unit Weight: 2 lbs.

Height: 34.4" tall when folded.

Does not adjust in height and is typically used by someone 5' - 5' 10".

Switch Stick Seat Stick is a versatile mobility aid that provides the user with a walking stick for support and a seat for resting. Ideal for attending a sporting event, concert or simply resting your legs after a long walk.

To Open: Hold the handle with one hand and push the seat down with the other hand.

Using the Stick: You MUST sit with the shaft of the stick facing you so that you can hold the handle in front of you for seat balance and safety.

Ferrules: If showing signs of wearing, the ferrules should be replaced as they become slippery.

Care of Your Stick: The shaft of the stick is printed aluminum and can scratch - therefore please take care and protect the 'paint work'. If the stick is wet, it is recommended that you dry it fully before putting it away.

